**REF FOLDY**

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***BRUGSANVISNING***

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# USER MANUAL

## 2-function double folding walking frame

Product code: FOLDY

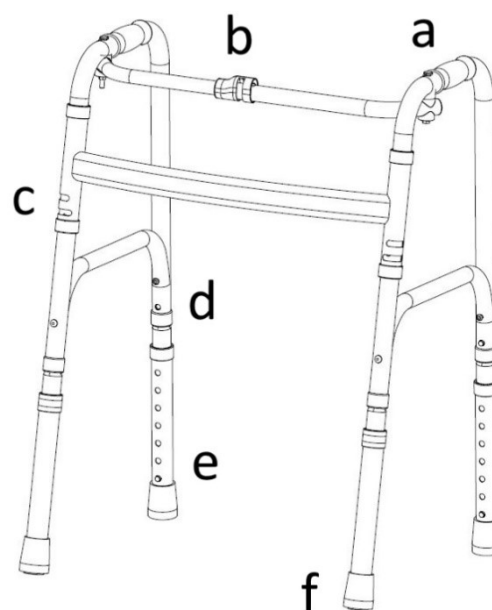
### 1. Characteristics

2-function folding walker is made of crosswise reinforced aluminium frame. The walker has 2 functions: the rigid and the reciprocating, which allows to move the sides alternatively without lifting the walker. The walker provides a wide area of support, as well as a long range of walking. Solid handgrips ensure a comfortable use. Owing to adjustable legs, the height of the walker can be adjusted according to user's needs. Rubber tips prevent the walker from sliding. Double folding function makes it easy to store and transport.

#### Elements of the walker:

- a. handgrip
- b. frame folding clip
- c. function change
- d. legs locking pin
- e. height adjustment
- f. anti-sliding rubber tip

### 2. Application

2-function walker is a support equipment designed for people with reduced mobility who need stable support and help while walking. It is also recommended to the elderly as an aid in keeping balance, improving walking comfort and ensuring safety. The use of the walker supports the recovery of people with lower limbs diseases and with locomotion or verticalization problems.

### 3. Assembly Unfolding

- To unfold the walker, pull the sides apart to the maximum position until the handgrips are locked.
- Then unfold the legs of the walker - pressing the legs locking pin, assemble the two parts of the tube, which are connected to each other by a rubber element.

### Folding

- To fold the walker, press the folding clip located in the middle on the crosswise frame and fold the sides.
- Then fold the legs of the walker - pressing the legs locking pin, disassemble the two parts of the tube, which are connected to each other by a rubber element.

### Height adjustment

- The height should be set on such a level that the user, who is leaning on the handles, stands upright, as far as he or she is able to, with arms in a resting position and elbows slightly bent. It is inadvisable to hang on the walker with all the body weight.
- To adjust the height of legs, press the metal pin, select appropriate height of the leg, and lock the position of the legs relative to each other. The legs are adjustable in 2.5 cm increments.

**Note!** The pins must be set on the same level in each leg.

### Selection of functions

- Rigid function - press the folding clip located in the middle of the frame, fold the legs slightly. Place the locking pins in the upper horizontal openings, used for function changing. Then pull the sides apart to the maximum position until the handgrips are locked
- Reciprocating function – press the folding clip located in the middle of the frame, fold the legs slightly. Place the locking pins in the lower horizontal openings, used for function changing. Then pull the sides apart to the maximum position until the handgrips are locked.

### 4. Technical data

Overall width	60 cm
---------------	-------

Inner width	52 cm
Overall depth	46 cm
Height adjustment	72-91 cm
Weight	2.35 kg
Maximum capacity	136 kg
Frame colour	black

## 5. Notes

**Timago International Group does not bear responsibility of improper use of the walker, failure to comply safety regulations and misuse.**

## 6. Maintenance

- Clean the frame of the walker with damp cloth using only water and mild detergents. Wipe dry after each use.
- **Do not use** for cleaning: strong polishing and abrasive agents, strong detergents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- Remove any dirt found on the moving parts of the walker (hinges, clamping screws, locks).

## 7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

## 8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

## 9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date

	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.  
 Learn more at: [www.timago.com](http://www.timago.com). Thank you for choosing Timago!

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