# USER MANUAL

## Steel 4-wheeled rollator

Product code: TGR-R RS 880

### 1. Characteristics

The folding 4-wheeled rollator is made of a powder-coated steel frame. It features an upholstered seat, a soft backrest, and a removable basket. The rollator is equipped with height-adjustable handles with anatomical handgrips, as well as brakes with a locking mechanism. It comes with 8" diameter wheels with a narrow cross-section.

**The set includes:** handles with brakes, frame with seat, backrest, basket.

#### Elements of the rollator:

- A. Push handle
- B. Brake
- C. Locking lever
- D. Brake cables
- E. Backrest
- F. Locking pin
- G. Side crossbar
- H. Hinge of side crossbar
- I. Strap for folding rollator
- J. Basket
- K. Upper adjustment of brake
- L. Lower adjustment of brake

### 2. Application

The rollator is designed for patients with reduced mobility who need stable support and help while walking. It is also recommended to the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the rollator affects positively on the return to activity of the people with locomotion and tilting to erect position problems. It guarantees better stabilization, support and security while walking.

### 3. Assembly

#### UNFOLDING AND FOLDING THE ROLLATOR

- To unfold the rollator, press down on the side crossbar located beneath the seat until the frame is fully opened. Then, insert the handles into the mounting openings and adjust them to the desired height by tightening the locking knobs. The brake cables should face forward. Position the seat onto the unfolded frame.
- Insert the backrest into the mounting openings at the front of the rollator until the locking pins engage.
- Hang the basket onto the frame.
- Remove the plastic covers from the wheels.
- To fold the rollator for transport, remove the basket, lower the seat, and pull upward on the strap connecting the side crossbar to the frame until the rollator is fully folded.

### **BRAKES FUNCTION**

- To stop the rollator, pull and hold the brake handle upwards.
- During a stop or while using the seat, lock the brake by pushing the brake handle down.
- To release the brake, pull the brake handle upwards.

### **HEIGHT ADJUSTMENT OF HANDLES**

The height of the handles should be adjusted so that the user can stand upright, as far as he or she is able to, with his or her shoulders in rest position and with slightly bent elbows. It is inadvisable to hang on the rollator with all the body weight.

- To adjust the height of the handles, unscrew the locking lever located on the outer side of the frame by, select appropriate height and tighten afterwards.

**NOTE!** The handles must be set on the same level.

### **BRAKES ADJUSTMENT**

- **Adjustment of the upper brake:** Unscrew the nut of the cable, pull it up to the appropriate length and lock it by tightening the locking screw.
- **Adjustment of the lower brake:** Loosen the mounting screw with the use of an open-end wrench, pull the black brake cable to adjust its tension, and then, tighten the mounting screw.

## **4. Technical data**

Overall width	60 cm
---------------	-------

Overall depth	73 cm
Seat dimensions	30 x 30 cm
Seat height from the ground	53 cm
Handles height adjustment	81-89 cm
Weight	8,5 kg
Maximum capacity	135 kg
Maximum capacity of the basket	5 kg
Frame colour	dark blue

## 5. Notes

**Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.**

## 6. Maintenance

### Cleaning

- Clean the frame with damp cloth and dry thoroughly.
- Do not use strong polishing agents due to the damage of protecting coatings.
- Clean the dirt from moving parts of the rollator regularly (hinges, clamping screws, locks).

## 7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

## 8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

## 9. Labels

	Reference number		Manufacturer
---	------------------	---	--------------

	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: [www.timago.com](http://www.timago.com). Thank you for choosing Timago!

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