REF TGR-R RS 8812

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BRUGSANVISNING

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USER MANUAL

Walking table with forearm support

Product code: TGR-R RS 8812

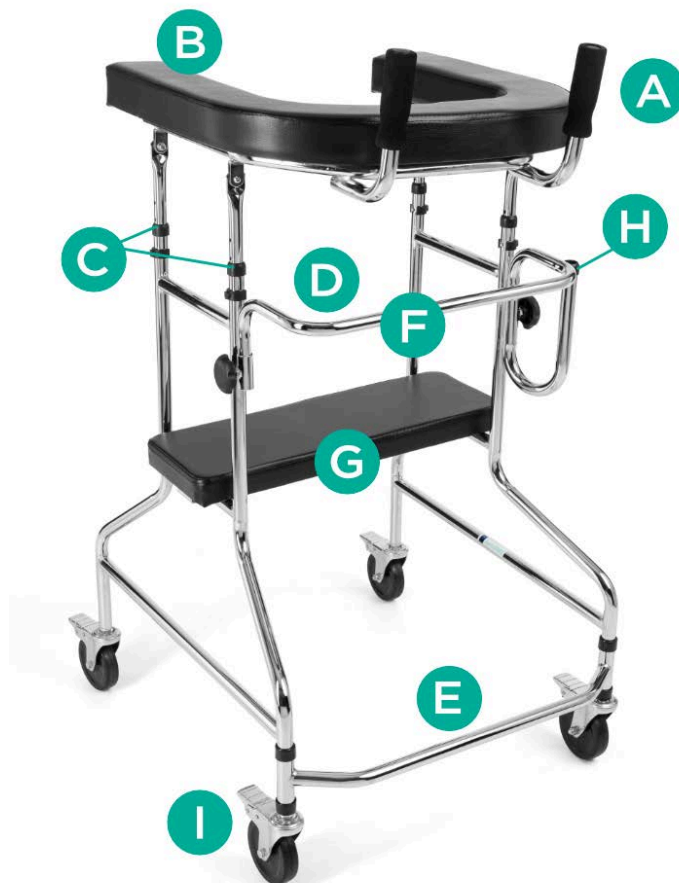
1. Characteristics

The walking table with forearm support is made of a chrome-plated steel frame. It features an IV stand holder and upholstered forearm supports with smooth height adjustment. The walking table is equipped with foam handgrips, swivel castors with brakes, and a removable seat.

The set includes: frame with castors, upper stabilizing frame, lower stabilizing frame, forearm support, seat, locking clips (4 pcs), mounting screws (2 pcs).

Elements of the rollator:

- A. Handgrip
- B. Forearm support
- C. Locking clip
- D. Upper stabilizing frame
- E. Lower stabilizing frame
- F. Mounting screw
- G. Seat
- H. Iv stand holder
- I. Brake (lock)

2. Application

The walking table is ancillary equipment designed for patients with reduced mobility who require stable support when transitioning to an upright position and during walking. It is particularly intended for individuals with various motor disorders, limited mobility or dysfunction and weakness in the lower limbs. The walking table is also recommended for elderly users as an aid in maintaining balance, improving walking comfort and preventing falls. Its use has positive effects on the return to activity for individuals with mobility and postural transition difficulties. It provides improved stability, support, and safety during walking.

3. Assembly

UNFOLDING

- Place the walking table on the ground. Fix the upper stabilizing frame in the mounting openings in the front part of the frame and tighten with mounting screws.
- Mount the forearm support into the mounting openings on the frame and secure it with locking clips on the chosen height.
- Fix the seat onto the lower frame part. Use the seat for temporary rest.

Note! While using the seat, all the brakes should be locked.

- When walking, the seat should be hung on the outer side of the frame.
- In case of using IV stand, place it in the IV stand holder in the front part of the frame.

HEIGHT ADJUSTMENT OF FOREARM SUPPORT

The height of the forearm support should be adjusted so that the user can stand as upright as possible, with shoulders relaxed and elbows resting comfortably on the supports. It is not advisable to place full body weight on the walking table.

- To adjust the height of forearm supports, pull out the locking clip which is located on the outer side of the frame, select appropriate height and fix the clip again.

4. Technical data

Overall width	63 cm
Overall depth	76 cm
Seat dimensions	56 x 20 cm
Seat height from the ground	54 cm
Forearm support height adjustment	101-124 cm
Weight	10,8 kg
Maximum capacity	100 kg

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Maintenance

Cleaning

- Clean the frame with damp cloth and dry thoroughly.
- Do not use strong polishing agents due to the damage of protecting coatings.
- Clean the impurities from moving parts of the walking table regularly (hinges, locking screws, brakes).

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!

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