**REF TGR-R RA 882**

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***BRUGSANVISNING***

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Product code: TGR-R RA 882

### 1. Characteristics

Folding 4-wheeled rollator is made of aluminium frame coated in paint with metallic lustre. The rollator has wide upholstered seat, soft backrest and removable basket. The product features height adjustable handles with anatomic handgrips, brakes with parking function and wheels with narrow cross section 8'' in diameter.

**The set includes:** two handles with brakes, frame with seat, backrest, basket, locking knobs – 2 pcs.

#### Elements of the rollator:

- A. Push handle
- B. Brake with parking function
- C. Locking knob
- D. Brake cables
- E. Backrest
- F. Locking pin
- G. Side crossbar
- H. Hinge of side crossbar
- I. Strap for folding rollator
- J. Basket
- K. Upper adjustment of brake
- L. Lower adjustment of brake
- M. Cane holder

### 2. Application

The rollator is designed for patients with reduced mobility who need stable support and help while walking. It is also recommended to the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the rollator affects positively on the return to activity of the people with locomotion and tilting to erect position problems. It guarantees better stabilization, support and security while walking.

### 3. Assembly

#### UNFOLDING AND FOLDING THE ROLLATOR

- To unfold the rollator, press the side crossbar, which is placed under the seat, until the hinges are completely unfolded. Fix the handles into the mounting openings and select appropriate height. Tighten the locking knobs afterwards. Brake cables should be placed in front. Lean the seat onto the frame.
- Fix the backrest into the mounting openings at the front of the rollator until the pins are locked.
- The basket can be hanged on the frame or fixed under the seat.
- Remove plastic casing from wheels.
- To fold the rollator for transport, lower the seat and pull upwards the strap which joins the side crossbar with frame until the frame is completely folded.

**NOTE!** Be careful while folding or dismantling the rollator due to the risk of smashing fingers in hinges.

#### BRAKES

- To stop the rollator, pull the brake handle upwards and hold.
- While stop or using the seat, lock the brake with parking function by pushing down the brake handle.
- To release the brake, pull the brake handle upwards.

#### HEIGHT ADJUSTMENT OF HANDLES

- To adjust the height of the handles, unscrew the locking knobs which are located on the outer side of the frame, select appropriate height and tighten the locking knobs.

**Note!** The handles must be set on the same level.

#### ADJUSTMENT OF BRAKES

- **Adjustment of upper brake:** unscrew the nut of the cable, pull it up to the appropriate length and lock it by tightening the locking screw.
- **Adjustment of lower brake:** loosen the mounting screw with the use of open-end wrench, pull the black brake cable to adjust the tension of the cable, and then, tighten the mounting screw.

### 4. Technical data

Overall width	60 cm
---------------	-------

Overall depth	66 cm
Seat dimensions	35 x 32 cm
Seat height from the ground	53 cm
Handles height adjustment	77-88 cm
Weight	8,2 kg
Maximum capacity	110 kg
Maximum capacity of the basket	5 kg
Frame colours	red, grey

## 5. Notes

**Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.**

## 6. Maintenance

### Cleaning

- Clean the frame with damp cloth and dry thoroughly.
- Do not use strong polishing agents due to the damage of protecting coatings.
- Clean the dirt from moving parts of the rollator regularly (hinges, clamping screws, locks).

## 7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

## 8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

## 9. Labels

	Reference number		Manufacturer
---	------------------	---	--------------

	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: [www.timago.com](http://www.timago.com). Thank you for choosing Timago!

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