USER MANUAL

Geriatric lift chair

Product code: RELAX

1. Characteristic

The RELAX geriatric lift chair provides comfortable rest for older adults living in their own homes. The chair is equipped with a dual independent drive that ensures maximum user comfort through a wide range of adjustment options. The handset allows the chair to be individually adjusted to the user's needs. Thanks to the lifting function, the chair tilts forward, making it easier for an elderly person to both stand up from and sit down in the chair.

The set includes: chair with base frame, backrest, power cable with power supply, handset.

2. Application


The geriatric chair is intended for people with:

- difficulties with independent mobility,
- mobility impairments and limitations,
- general weakness resulting from advanced age.

3. Assembling, disassembling and operation

BEFORE ASSEMBLING

- Make sure that the chair is placed on a level surface.
- Make sure there is enough space between the chair, the wall and other objects in the room.
- Remove the restraints securing the chair wires to the base frame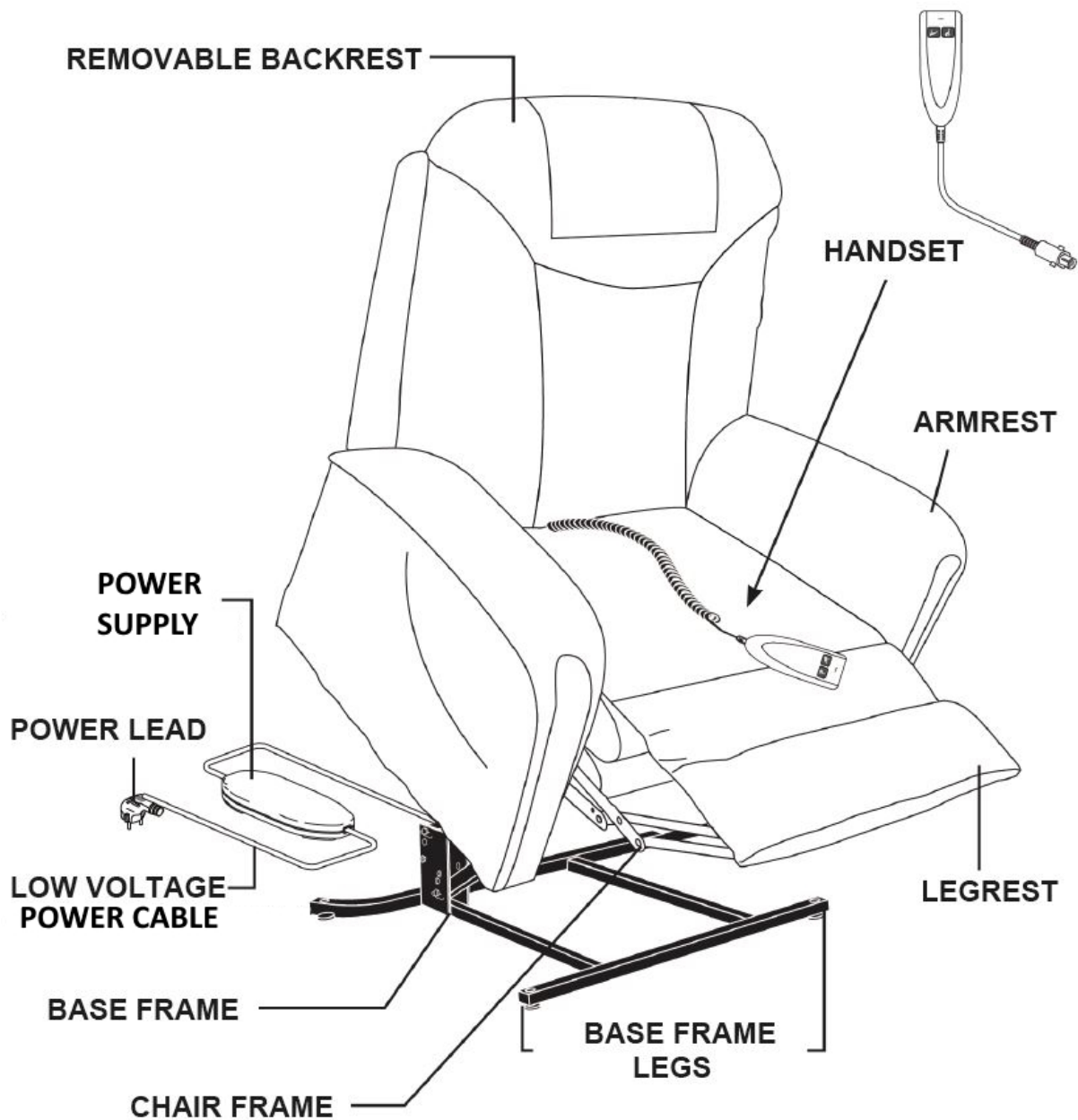

ASSEMBLING

- The chair comes fully assembled, except for the removable backrest.
- Connect the backrest to the seat using the two mounting rails located on both sides of the seat.
- Press down on the backrest until both mounting rails snap into place.
- Make sure the backrest is securely fastened by pulling it slightly upward. If the backrest feels loose, repeat the assembly steps above.

- Use the front and rear Velcro straps to fasten the two fabric covers that connect the backrest to the seat.
- Connect the power cable to the socket located on the base frame, then lock the socket.
- Connect the handset cable to the socket located in the side pocket of the chair, then lock the socket.

DISASSEMBLING

- After unlocking the socket locks, disconnect both the handset and power cable plugs.
- Unfasten both fabric covers (front and rear Velcro straps).
- Using an appropriate tool, gently bend the locking tabs on both mounting rails and slide the backrest out.

HANDSET OPERATION

Button A

Downward movement of the backrest

Button B

Upward movement of the backrest


Button C

Lifting - upward movement of the entire chair until it tilts slightly to make it easier to stand up from the chair. If the footrests are raised, pressing the button at first lowers the footrests, and then lifting the chair.

Button D

Leveling - downward movement of the entire chair.


After reaching the neutral position, further pressing the button raises the footrest.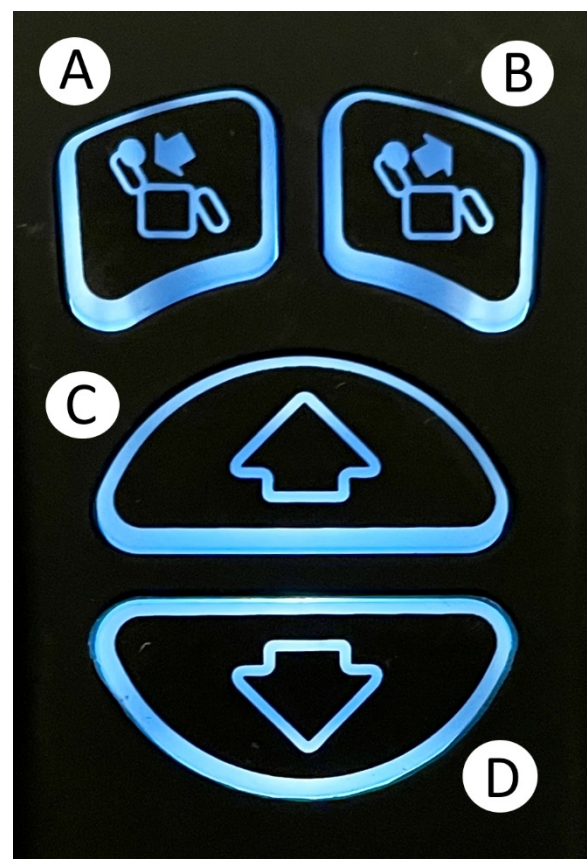

CHAIR TRANSPORTATION

The chair is equipped with two rear wheels for transportation (figure below).

To move the seat, tilt the backrest slightly backward so that the chair rests on the wheels. Then move the chair.

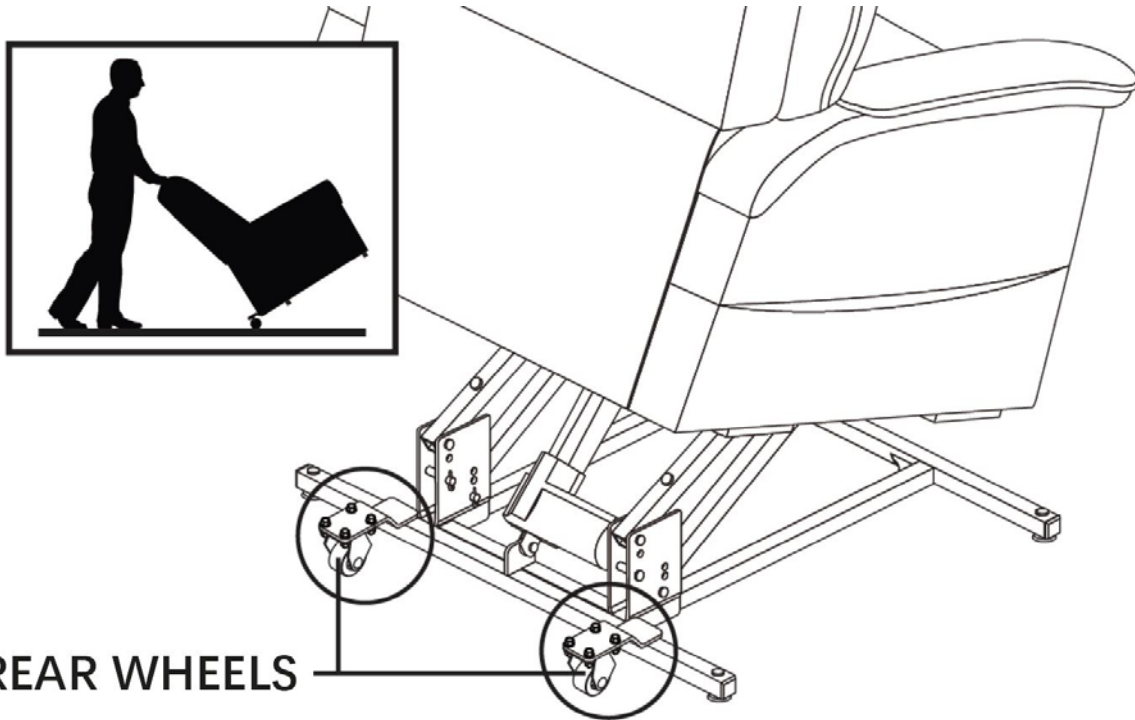4. Technical data

Total length (sit position)	85 cm
Total height (sit position)	109 cm
Seat depth	43 cm
Backrest height	76 cm
Seat height (sit position)	51,5 cm
Total width	80 cm
Total height	55 cm
Armrest height	12,5 cm
Total length (recline position)	159 cm
Total height (recline position)	89 cm
Total length (lifting position)	84 cm
Total height (lifting position)	129 cm
Backrest recline angle	141°

Footrest rise angle	89°
Chair lift angle	17°
Weight	43 kg
Max capacity	130 kg
Drive type	dual
Upholstery color	Mocca

5. Notes and remarks

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

- Chair is designed for indoor use.
- Sitting on the chair by more than one person is prohibited.
- Do not sit on the backrest, armrest or footrest of the chair.
- Keep hands and feet away from moving parts of the base frame and chair frame.

6. Maintenance

Clean the upholstery with a damp cloth and then dry thoroughly. Do not use harsh disinfectants, as this risks damaging the fabric. Remove dirt from the moving parts of the base frame and chair frame.

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		UDI code

	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!

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