# **REF** SLIDE-TIM

EN	SLIDE SHEET
PL	ŁATWOŚLIZG
CS	KLUZNÝ PROSTĚRADLO
DA	GLIDELAGEN
DE	GLEITLAKEN
EL	ΣΕΝΤΟΝΙ ΟΛΙΣΘΗΣΗΣ
ET	LIBISEV LINA
HR	KLIZAJUĆA PLAHTA
HU	CSÚSZÓLEPEDŐ
IT	LENZUOLO SCIVOLANTE
LT	SLYDIMO PAKLODĖ
LV	SLĪDOŠA PALAGS
PT	LENÇOL DESLIZANTE
RO	CEARȘAF DE ALUNECARE
SK	KLZAVÁ PLACHTA
SL	DRSNA RJUHA
SV	GLIDLAKAN

# USER MANUAL

## Slide sheet

Product code: SLIDE-TIM

### 1. Characteristics

Slide sheet is made of durable polyester coated with waterproof silicone. Thanks to its cylindrical structure and low-friction material it allows for easy and safe sliding of patients, without excessive physical effort or strain on the caregiver's body.

### 2. Application

The slide sheet is designed for use in medical facilities and home environments. It can be used to assist in moving patients on different surfaces. The use of the slide sheet has a positive effect on caregiver safety, effectively reducing the risk of back and muscle strain, and enhancing patient comfort thanks to reduced friction, which helps prevent bedsores.

### 4. Technical data

Available sizes	72 x 71 cm / 122 x 71 cm
Thickness	0,6 mm
Material	100% polyester

### 5. Notes

**Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.**

### 6. Maintenance

Slide sheet can be machine washed (on delicate cycle) with use of mild detergents. Do not use fabric softeners. Wash separately.

- **Do not** use bleach, conditioners or stain removes on the product. Do not iron.

## 7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

## 8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please keep in mind that for warranty purposes the proof of purchase (receipt or invoice) should be retained.

## 9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: [www.timago.com](http://www.timago.com). Thank you for choosing Timago!

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