USER MANUAL

Toilet safety rail

Product code: JMC-C 5300KD

1. Characteristics


Toilet safety rail is made of ultra-light anodized aluminium. It is height and width adjustable in order to meet user needs.

The set includes:

- Side rails (2 pcs)
- Connecting bar (2 pcs)
- Height adjustable legs with anti-slippery tips (2 pcs)
- Mounting element - connector

Elements of the safety rail:

1. Height adjustable leg (2 pcs)
2. Side rail (2 pcs)
3. Connecting bar (2 pcs)
4. Connector
5. Anti-slippery tip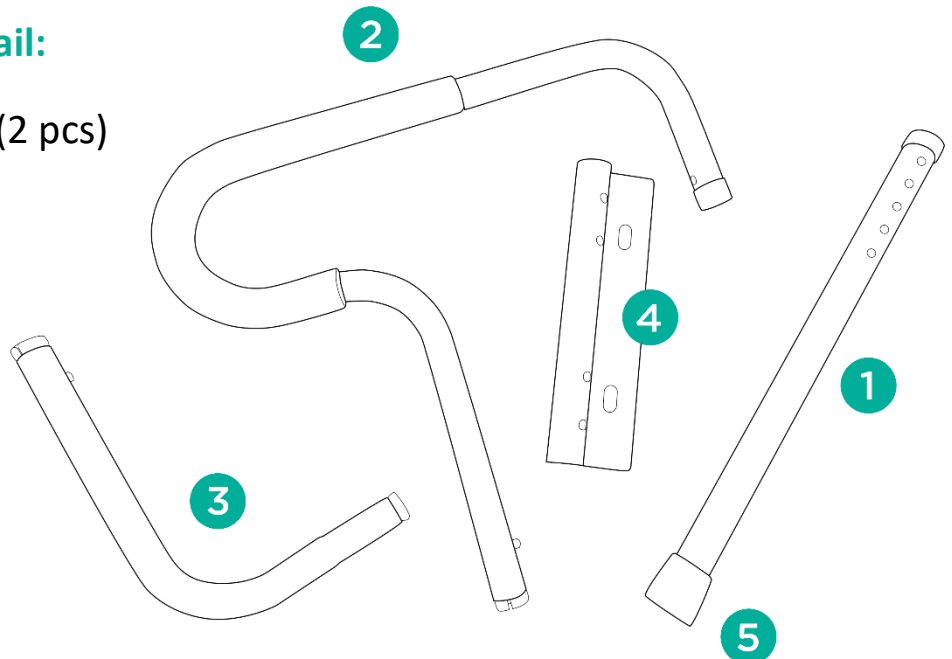


2. Application

Toilet safety rail is an ancillary equipment which helps the user to perform physiological activities. The rail is recommended to people with dysfunctions and for those who lost the ability of unassisted sitting down and standing up. Owing to height and width adjustment, the frame can be adjusted to the size of the toilet.

3. Assembly

Step 1. Remove the toilet seat; unscrew mounting screws on the toilet bowl.

Step 2. Assemble the frame by connecting the parts with locking pins (picture 1).

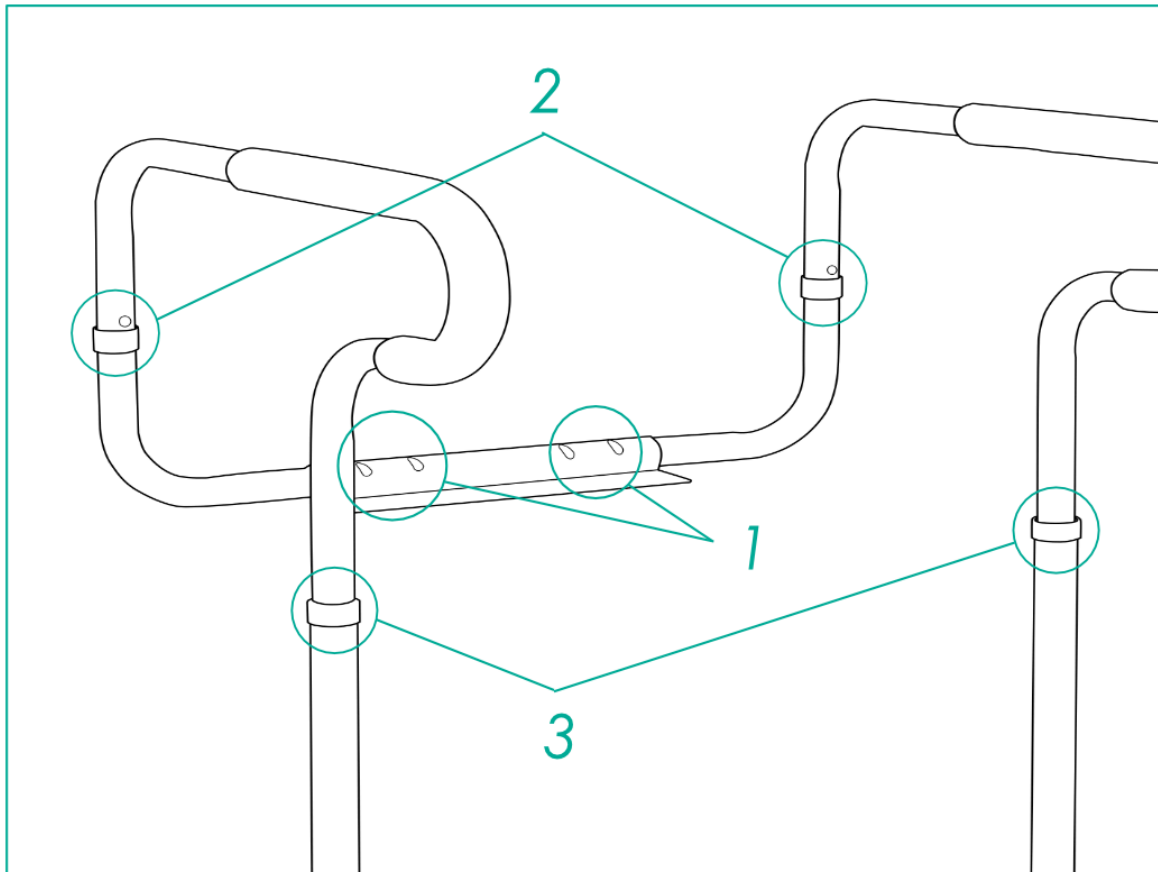Picture 1

Step 3. Place the connector onto the mounting openings of the toilet bowl.

Step 4. Adjust the height by pressing the locking pin located at the legs, select appropriate height and lock the position of the tubes relative to each other. To adjust width of side frames use pins located on the connector.

Note! The pins must be set on the same level in each leg. Legs should stand stably on the ground.

Step 5. Reinstall the toilet seat.

4. Technical data

Overall width	57-65 cm
Overall depth	47 cm
Height	73-83 cm
Length between mounting openings on connector (4)	11,8-16,2 cm
Weight	1,40 kg
Maximum capacity	136 kg

Frame colour	silver
--------------	--------

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Maintenance

Reviews

Systematically make visual inspection of all the joining elements (clamps, hinges) including:

- the condition of locking pins
- stability and look of the frame

Cleaning

Control the technical condition of the toilet safety rail regularly:

- Clean the rail regularly using only water and mild detergents. Dry thoroughly.
- Do not use for cleaning: strong polishing agents and detergents, abrasive agents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device

	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.
Learn more at: **www.timago.com**. Thank you for choosing Timago!

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