USER MANUAL

Bathtub bench

Product code: FS 794L-1

1. Characteristics

The bathtub bench features a seat and a profiled frame with anti-slip tips that prevent the bench from sliding on the bathtub, providing safety and stability during use. Thanks to the adjustable frame, the bench can fit various bathtub widths, while the PVC seat and aluminium frame ensures long-term durability. The product is made of moisture-resistant materials.

The set includes: seat, two elements of side frame

Elements of the bench:

- A. Seat
- B. Locking pin
- C. Anti-sliding tips

2. Application

The bathtub bench is ancillary equipment which helps the user to perform hygienic activities. The bench is recommended to people with reduced mobility which prevents or hinders the unassisted performance of daily living activities.

3. Assembly

Step 1. Mount side frame elements one after another on both sides.

Step 2. Adjust the width using locking pin and lock side frame element.

Step 3. Make sure that all the elements are fixed correctly and side frame is adjusted properly.

Note! The frame should be adjusted to the bath width correspondingly. Anti-slip tips should be placed firmly on the edges of the bath.

4. Technical data

Overall width	73-78 cm
Inner width	47-52 cm
Depth of frame	14 cm
Seat dimensions	41 x 23 cm

Weight	1,8 kg
Maximum capacity	100 kg

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Maintenance

- Clean the bathtub bench regularly using only water, sponge and mild detergents for plastic materials such as soap. Dry thoroughly.
- **Do not use** for cleaning: strong detergents, abrasive agents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- At least once every three months check:
 - condition of locking pins for height adjustment.
 - condition of tubes (if cracks).

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

REF	Reference number		Manufacturer
LOT	LOT number	UDI	UDI code
SN	Serial number	MD	Medical device

	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: **www.timago.com**. Thank you for choosing Timago!

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