USER MANUAL

Folding blind cane

Product code: TGR-R LA 853N

1. Characteristics


The blind cane is made of four aluminum sections connected by a strong, double elastic cord. The shaft is equipped with a rotating, interchangeable tip. Reflective straps enhance safety by making it easier for drivers and other road users to identify a visually impaired person.

Elements of the cane:

- A. handgrip
- B. wrist strap
- C. reflective band
- D. rotary tip

2. Application

The cane is designed for the blind and visually impaired. The cane enables the user to identify the terrain independently. It protects from bumping into obstacles which are placed on the road and it signals user's disability to other people.

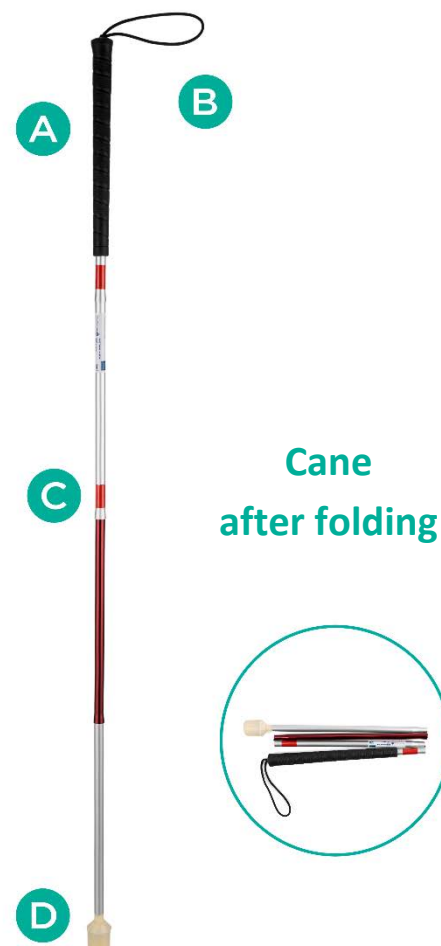3. Technical data

Type	4 sectional
Colour	silver
Height	122 cm
Folded length	36,5 cm
Handgrip	rubber with wrist handgrip
Maximum capacity:	100 kg
Weight	0,2 kg

4. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

5. Cleaning and maintenance

- Clean the cane using only water and mild detergents, dry thoroughly.
- **Do not use:** strong detergents, abrasive agents, bleach, petrol nor solvents (it may cause frame damage, visible defects, and in consequence – loss of warranty rights).
- It is necessary to check the cord regularly at the point of sections joint. In case of abrasion, cracking or tearing, discontinue using the cane.
- It is recommended to check rubber tip regularly. Replace worn tip immediately.

6. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

7. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction

The manufacturer checked for compliance with essential medical devices requirements.

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!

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