REF **CARBON**

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BRUGSANVISNING

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Product code: CARBON

1. Characteristics

Folding 4-wheeled rollator is made of aluminium frame coated in paint with metallic lustre. The rollator is equipped with wide upholstered seat, soft backrest and basket. It has height adjustable handles with anatomic handgrips, parking brakes and wheels with narrow cross section 8" in diameter.

Elements of the rollator:

- A. Handgrip
- B. Parking brake
- C. Fixing knob
- D. Brake cables
- E. Backrest
- F. Locking pin
- G. Crossbar
- H. Crossbar hinge
- I. Basket
- J. Upper brake adjustment
- K. Lower brake adjustment

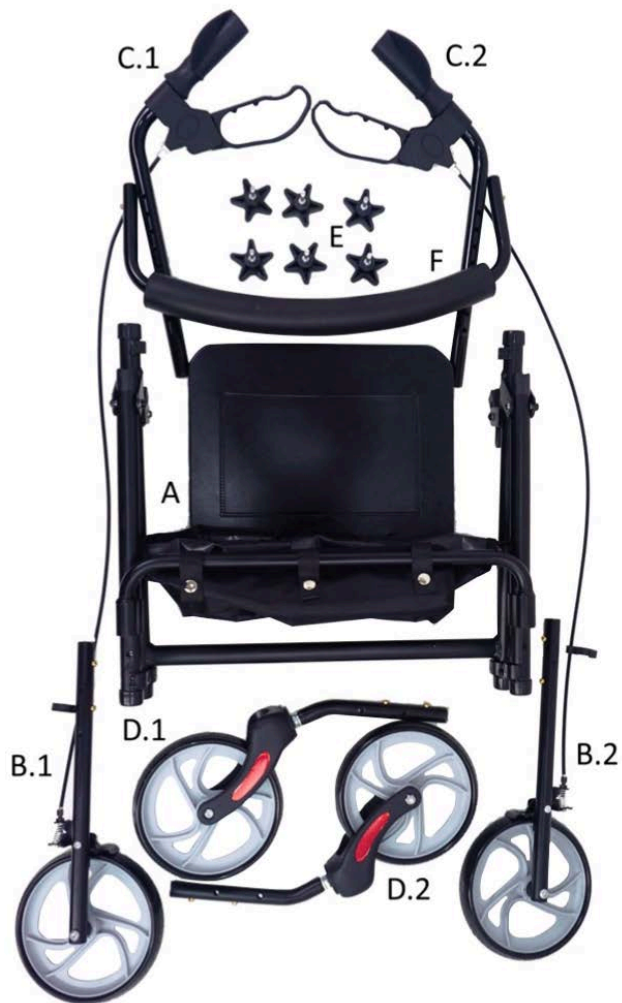
2. Application

The rollator is designed for patients with reduced mobility who need stable support and help while walking. It is also recommended to the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the rollator has a positive effect on the return to activity of people with locomotion and verticalization problems. It guarantees better stabilization, support and security while walking.

3. Assembly and adjustment

Mounting elements:

- A. Frame
- B.1 Left rear leg
- B.2 Right rear leg
- C.1 Left handle
- C.2 Right handle
- D.1 Left front leg
- D.2 Right front leg
- E. Fixing knobs
- F. Backrest

4. Assembly

1. Unfold the frame (A) by laying both hinges of the crossbar flat. The backrest mounting sockets indicate the front of the frame/rollator.
2. Insert the left rear leg (B.1) into the left rear hole of the frame by pressing the locking pin (pin facing outward). Lock the pin into one of the two frame holes, depending on the desired height of the rollator. Secure the leg by tightening the fixing knob (E).
3. The left rear leg (B.1) is connected to the left handle (C.1) by a brake cable. Insert the left handle into the corresponding hole in the frame and adjust it to the desired height. Route the brake cable along the outside and secure it in the plastic clip on the frame. Lock the handle by tightening the fixing knob (E).
4. Repeat the same steps to install the right rear leg (B.2) and right handle (C.2), making sure to tighten the fixing knobs (E).
5. Insert the left front leg (D.1) into the left front hole of the frame by pressing the locking pin. Lock the pin in one of the two height holes, then secure the leg by tightening the fixing knob (E). Note: All four legs must be set to the same height.
6. Repeat the same steps to install the right front leg (D.2), ensuring the fixing knob (E) is properly tightened.

7. Mount the backrest (F) by securing both locking pins in place.

FOLDING AND UNFOLDING THE ROLLATOR

- To fold the assembled rollator, lift the seat and pull up on the crossbar located under the basket. Then fold the rollator frame downward.
- To unfold the rollator, open the frame and push down on the crossbar beneath the basket until the crossbar hinges are fully extended. Finally, lower the seat into position.

BRAKES FUNCTION

- To stop the rollator, pull and hold the brake handle upwards.
- During a stop or while using the seat, lock the brake by pushing the brake handle down.
- To release the brake, pull the brake handle upwards.

HANDLES HEIGHT ADJUSTMENT

- To adjust the height of the handles, unscrew the locking knobs located on the outer side of the frame, select the appropriate height and tighten the locking knobs.

Note! The handles must be set on the same level.

BRAKES ADJUSTMENT

- **Adjustment of the upper brake:** Unscrew the nut of the cable, pull it up to the appropriate length and lock it by tightening the locking screw.
- **Adjustment of the lower brake:** Loosen the mounting screw with the use of an open-end wrench, pull the black brake cable to adjust its tension, and then, tighten the mounting screw.

5. Technical data

Overall width	64 cm
Overall depth	72 cm
Seat width	45 cm
Overall height	77-95 cm
Seat height from the ground	49,5/54,5 cm
Handles height adjustment	77-88 cm
Weight	7,2 kg
Maximum capacity	100 kg

Maximum capacity of the basket	5 kg
Frame colours	black

6. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

7. Maintenance

Clean the frame with damp cloth and dry thoroughly. Do not use strong polishing agents due to the damage of protecting coatings. Clean the dirt from moving parts of the rollator regularly (hinges, clamping screws, locks).

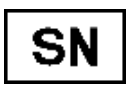
8. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

9. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

10. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!

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