REF QUATRO

- EN ALUMINIUM QUAD CANE
- PL CZWÓRNÓG INWALIDZKI ALUMINIOWY
- CS HŮL ČTYŘBODOVÁ
- DA FIREPUNKTSSTOK, 4-PUNKTSSTOK
- DE VIERBEINER CANE
- EL ΜΠΑΣΤΟΥΝΙ ΤΕΤΡΆΠΟΔΟ
- ET HARKKEPP 4 TOETUSPUNKTIGA
- HR ALUMINIJSKI ŠTAP S 4 NOGE
- HU ALUMÍNIUM NÉGYES BOT
- IT BASTONE QUADRUPLO IN ALLUMINIO
- LT ALIUMININĖ LAZDELĖ SU 4 KOJELĖMIS
- LV KVADRIPODS AR
- PT BENGALA 4 PONTAS
- RO BASTON TETRAPOD
- SK ODĹAHČENÁ 4-BODOVÁ PALICA
- SL PALICA ŠTIRINOŽNA ALUMINIJASTA
- SV QUAD KÄPP I ALUMINIUM

USER MANUAL

Aluminium quad cane

Product code: QUATRO

1. Characteristics

The quad cane is made of anodized aluminium frame. It is equipped with ergonomic handles, wrist strap, height adjustment and a base with four legs.

2. Application

The quad cane is an ancillary equipment which is designed for patients with reduced mobility who need stable support and help while walking. It is also recommended to the elderly as an aid in keeping balance. The design with four legs provides better stability than regular canes. Additional handgrip helps user when standing up, providing better stability.

Contraindications: coordination and balance disorders, advanced rheumatic changes, injuries and dysfunctions of upper limbs which may hinder the support on the quad cane.

3. Adjustment

- To adjust the quad cane, unscrew the adjustment lock and press the blocking pin to select appropriate height. Tighten the adjustment lock. The quad cane is adjustable in 2.5 cm increments.
- To set the proper length of the quad cane, stand upright and bent upper limb in elbow joint at the angle of 20°.
- The quad cane should be adjusted individually to the user so that the upper handgrip is at hip joint level.
- Before use, ensure that the quad cane is stable and adjustment lock is tighten.

NOTE! In case of doubts concerning the technique of moving with the use of quad cane, consult a person who has appropriate knowledge or qualifications.

4. Technical data

Type	adjustable, aluminium
Handle	black, foam
Base dimensions	20 x 17,5 cm
Height adjustment	75-92 cm
Weight	1 kg
Max. capacity	136 kg
Base color	black
Shaft color	blue

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Cleaning and maintenance

- Clean the quad cane using water and mild detergents only.
- **Do not use:** strong detergents, abrasive agents, bleach, petrol nor solvents (it may cause frame damage, visible defects, and in consequence – loss of warranty rights).
- It is recommended to check rubber tips regularly. Replace worn tips immediately.

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!

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