USER MANUAL

Aluminium adjustable cane, foldable

Product code: JMC-C 2052

1. Characteristics

The cane is made of an aluminum shaft with a wood-like handgrip and a soft, non-slip rubber tip. Thanks to the locking-pin height adjustment and the ability to fold it into three parts, the cane is highly convenient for everyday use.

Elements of the cane:

- A. handgrip
- B. wrist strap
- C. height adjustment – locking pin
- D. rubber tip

2. Application

The cane is ancillary equipment which is used to increase the area of support in case of patients with reduced mobility. It is also recommended to patients who need help and support while walking and it can also be used as an aid in keeping balance after the injuries of lower limbs or trunk.

Contraindications: coordination and balance disorders, advanced rheumatic changes, injuries and dysfunctions of upper limbs which may hinder the user to support on the cane.

3. Adjustment

To adjust the crutch, press the metal pin and select the appropriate position of the inner tube. Lock the tubes in place using the metal pin. The crutch is adjustable in 2.5 cm increments.

- To set the proper length of the cane, stand upright and bent upper limb in elbow joint at the angle of 20°.

- The cane should be adjusted individually to the user so that the handgrip is at hip joint level.
- Before use, ensure that the crutch is stable and that the locking pin is fully protruding through the adjustment hole.

NOTE! In case of doubts concerning the technique of moving with the use of cane, consult a person who has appropriate knowledge or qualifications.

4. Technical data

Type	foldable
Height adjustment	78-88 cm
Maximum capacity	90 kg
Cane weight	0,38 kg
Colours	brown, black

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Cleaning and maintenance

- Clean the cane using only water and mild detergents, dry thoroughly.
- **Do not use:** strong detergents, abrasive agents, bleach, petrol nor solvents (it may cause frame damage, visible defects, and in consequence – loss of warranty rights).
- It is recommended to check rubber tip regularly. Replace worn tip immediately.

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!

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