# USER MANUAL

## Aluminium 4-wheeled rollator

Product code: LORENZA


### 1. Characteristics

Four-wheeled rollator is made of high-quality aluminium. It has solid front 10" and rear 7,5" diameter wheels, height-adjustable handles finished with anatomical grips, brakes with parking function, seat with backrest and bag. The structure is based on a crossover frame and an oval cross-section of tubes, thanks to which the resistance to axial pressure is significantly increased. The hidden brake cable provides an aesthetically pleasing appearance.

**The set includes:** frame with seat, front and back wheels, backrest, shopping bag.

### Elements of the rollator: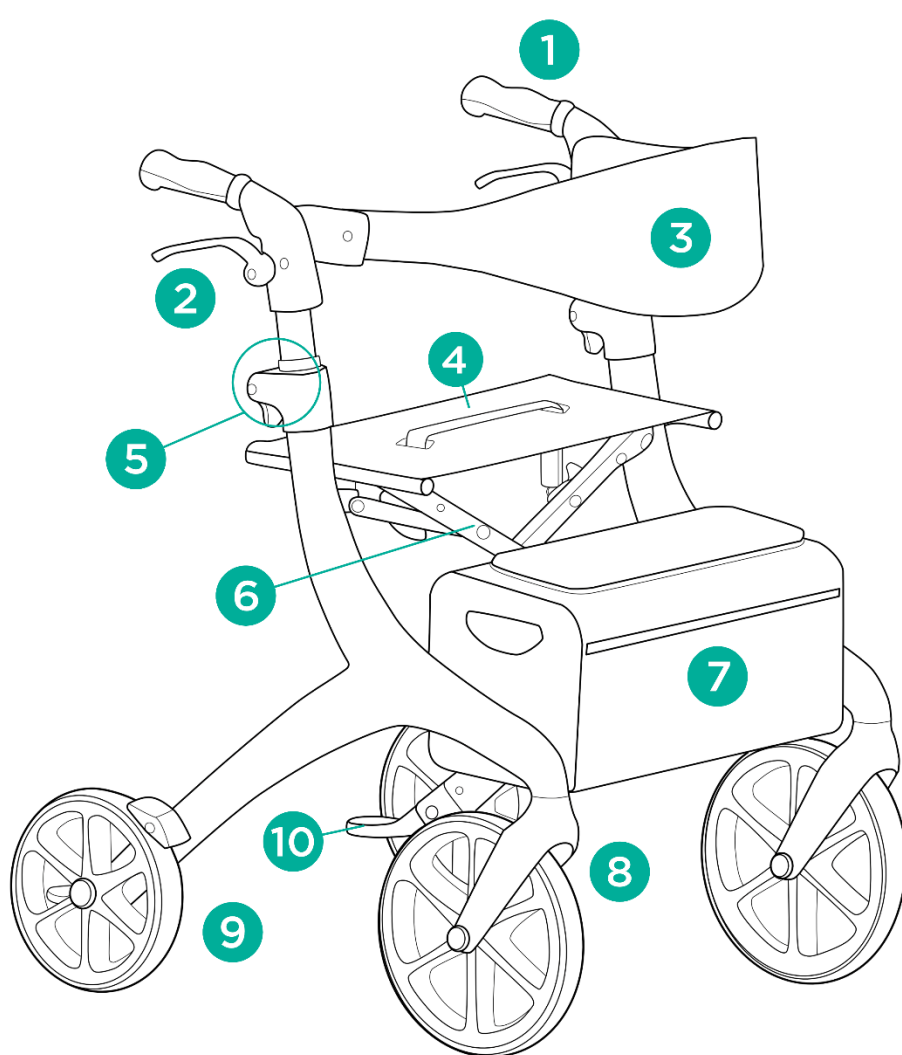

- 1 handgrip
- 2 brake with parking handle
- 3 backrest
- 4 seat
- 5 height adjustment lock
- 6 crossbar
- 7 bag
- 8 front wheel 10"
- 9 rear wheel 7,5"
- 10 support pedal

### 2. Application

The rollator is designed for patients with reduced mobility who need stable support and help while walking. It is also recommended to the elderly as an aid

in keeping balance in order to improve walking comfort and ensuring safety from falling. The use of the walker has a positive effect on the return to activity of people with locomotion and tilting to erect position problems. It guarantees better stabilization, support and security while walking.

### **3. Assembling**

#### **Unfolding**

- To unfold the rollator, grab both handgrips and pull them outwards. Press the seat gently on both sides to fully unfold the rollator (characteristic “click” sound).

#### **Folding**

- To fold the rollator, pull the seat up.
- Ensure that rollator is correctly folded.

#### **Handgrips height adjustment**

To adjust the height of the handgrip:

- Press the brake down to activate the parking brake and immobilize the rollator.
- Lift the adjustment clamp up.
- Adjust the height of the handle.
- Tighten the clamp to lock the adjusted height of the handles. Pull the handle up and down several times to check that it is locked correctly.
- Adjust the height of the second handgrip in a similar way.

**NOTE!** Both handgrips should be set at the same height.

#### **Brake function**

- To stop the rollator, pull the brake handle up, then hold.
- When not in motion or using the seat, lock the parking brake with the parking clamp by pulling the brake handle down.
- To release parking brake, pull the handle upwards.

#### **Removing and attaching backrest and bag**

Bag and backrest can be dismounted.

- To remove backrest, press the button located at the top, inner part of the frame and pull the backrest up.
- To remove the bag, pull it up from mounting openings located on the bottom, inner part of the frame.
- To attach bag and backrest, repeat steps backwards.

## Support pedal

- The support pedal is used to help overcome obstacles, such as curbs. Press down on the pedal with your foot to lift the rollator up.

## 4. Technical data

Dimensions after folding	70 x 15 x 86,5 cm
Overall width after unfolding	58 cm
Overall length	87 cm
Seat width	45 cm
Seat height from the ground	55 cm
Handgrip height adjustment	86,5-96,5 cm
Rollator weight	7,50 kg
Maximum capacity	136 kg
Frame colour	dark blue

## 5. Notes

**Timago International Group does not bear responsibility of improper use of the rollator, failure to comply safety regulations and misuse.**

## 6. Maintenance

- Clean the frame with damp cloth and dry thoroughly.
- Do not use strong polishing agents due to the risk of damage of protecting coatings.
- Clean the dirt from moving parts of the rollator regularly (hinges, clamping screws, locks).

## 7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

## 8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

## 9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.

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