# USER MANUAL

## Walking table with forearm support

Product code: PAULA

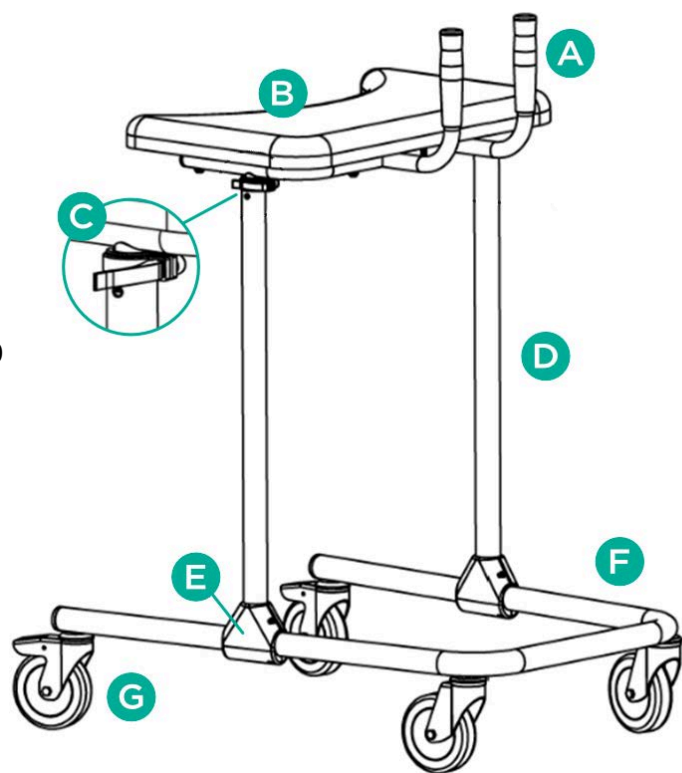
### 1. Characteristics

Walking table with forearm support is made of lacquered steel frame. It is equipped with rubber handgrips, upholstered forearm support, smooth height adjustment and swivel castors with locks on rear wheels. Thanks to its foldable construction, it's easy to store and transport.

**The set includes:** bottom frame with wheels, top frame (2 pcs), forearm support, locking screws (4 pcs), screw washers (4 pcs), mounting clips (2 x 2 pcs), assembly key.

#### Elements of the walking table:

- A. Handgrip
- B. Forearm support
- C. Height adjustment locking clasp
- D. Top frame
- E. Mounting clip
- F. Bottom frame
- G. Castor wheel with brake


### 2. Application

The walking table is ancillary equipment and is designed for patients with reduced mobility that need stable support and help while tilting to erect position and walking. It is especially designed for patients who suffer from different types of motor disorders together with limited mobility, dysfunction or weakening of lower limbs. The walking table is also recommended to the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the walking table has positive effects on the return to activity of the people with movement and tilting to erect position problems. It guarantees better stabilization, support and security while walking.


### 3. Assembly

#### Unfolding

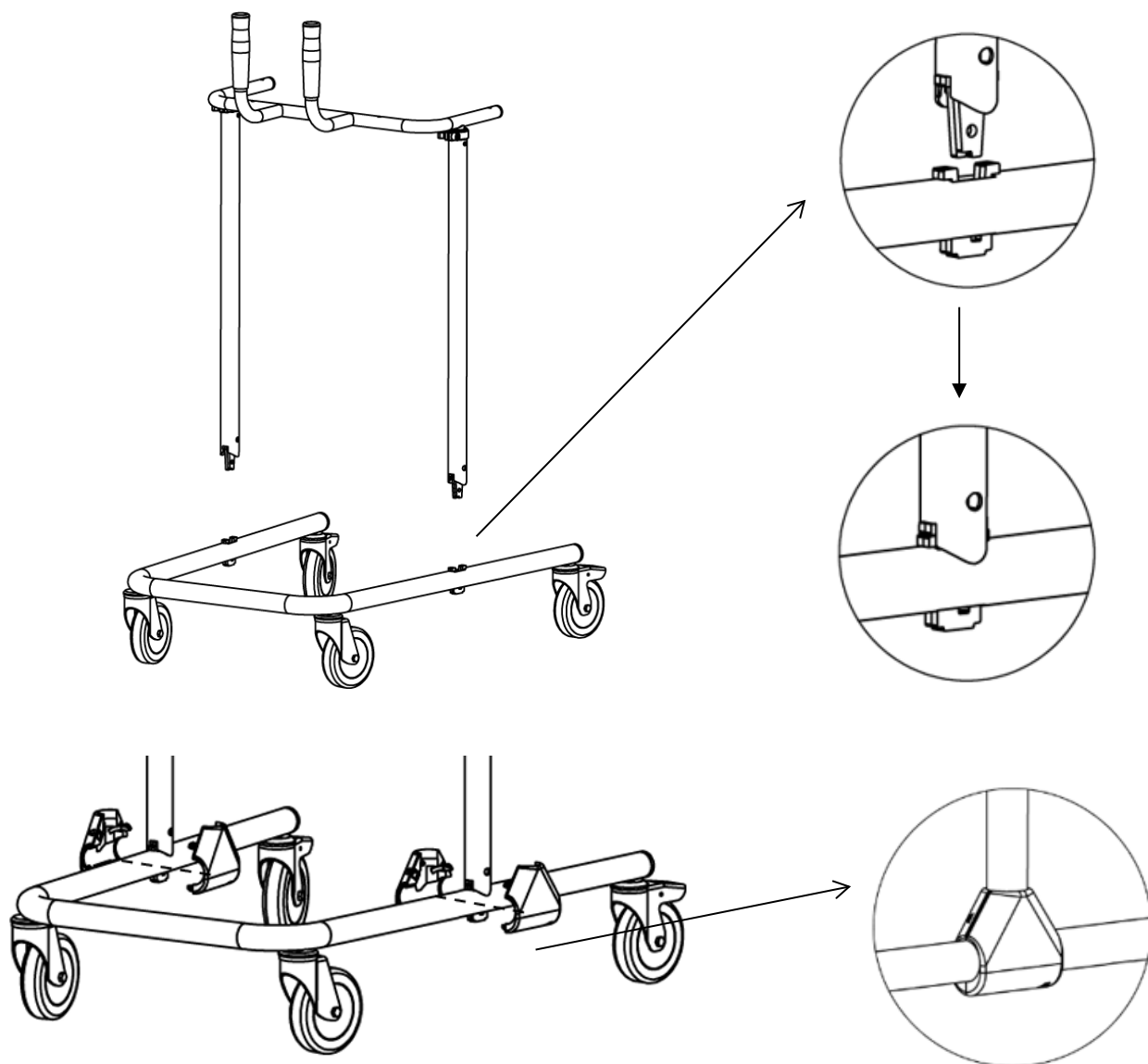1. Place the bottom frame with castors on the ground.
2. Put top frame tubes into bottom frame openings (picture 1).
3. To connect bottom and top frame use special mounting clips on both sides (picture 2).
4. To secure the height before mounting forearm support, tighten clasp by using handle, simultaneously holding nut on the other side with proper tool (recommended: adjustable wrench). After that lock clasp by pushing handle towards the frame (picture 5 & 6).
5. Mount forearm support on top of the two frame parts (with handles) using screws and screw washers. Tighten them with assembly key (Allen key) that comes in set (picture 3).

Picture 1 & 2

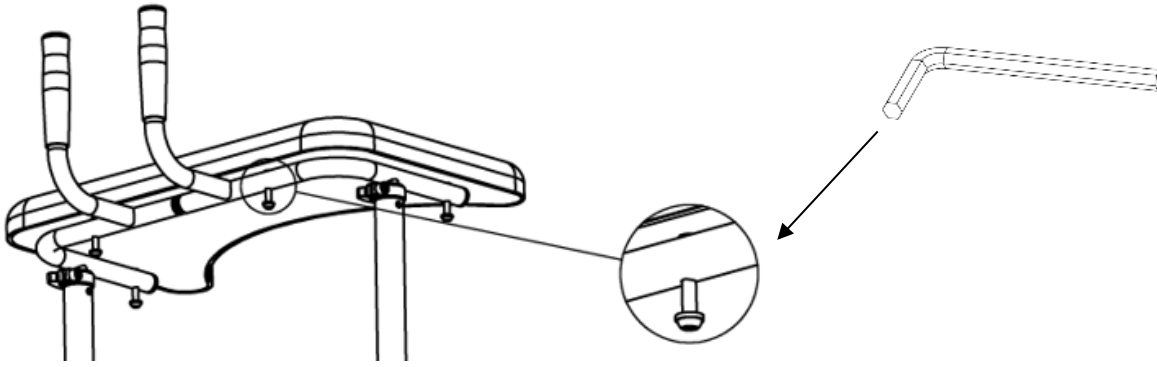Picture 3

## LOCK FUNCTION

Press lock pedal to lock forearm support in place. To unlock support, press locked pedal (picture 4).



Picture 4

## HEIGHT ADJUSTMENT OF FOREARM SUPPORT

The height of forearm support should be set so that the user can stand upright, as far as he or she is able to, with his or her shoulders in rest position and with elbows leaned against the support. It is inadvisable to hang on the walking table with all the body weight.

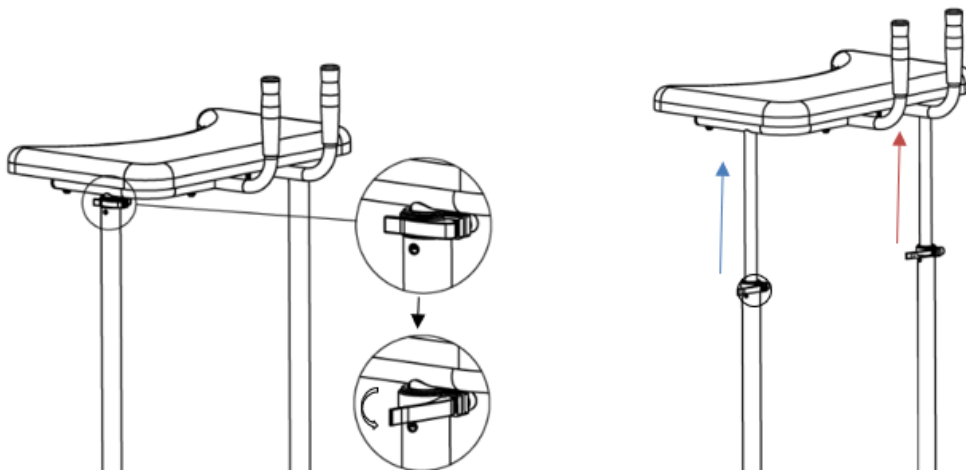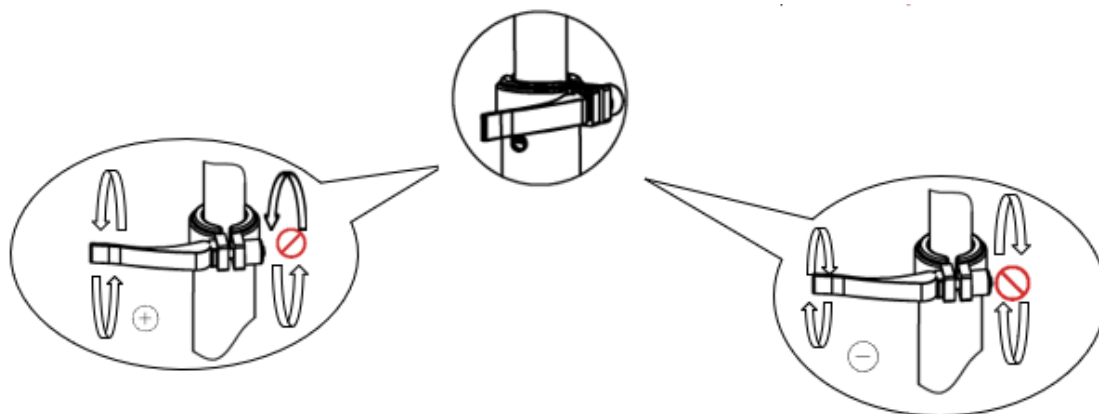- To adjust the height of forearm supports unlock the locking clasp. After selecting appropriate height, tighten lock by using clasp handle, simultaneously holding nut on the other side with proper tool. After that lock clasp by pushing handle to a frame (picture 5 & 6). Repeat on the other side.

Picture 5 & 6

#### 4. Technical data

Overall width	55 cm
Overall length	66,5 cm
Forearm support dimensions	51 x 31 cm
Forearm support height adjustment	78-129 cm
Weight	16 kg
Maximum capacity	180 kg

#### 5. Notes

**Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.**

#### 6. Maintenance

##### Cleaning

- Clean the frame with damp cloth and dry thoroughly.
- Do not use strong polishing agents due to the damage of protecting coatings.
- Clean the impurities from moving parts of the walking table regularly (hinges, wing screws, locks).

#### 7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

## 8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

## 9. Medical device incidents

Any serious incident involving a medical device must be reported to the manufacturer and to the competent authority of the European Union member state where the user or patient resides.

## 10. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The product meets the legal requirements of the European Union for medical devices (MDR 2017/745).		

Our complete line of products is available from our distributors. Learn more at: [www.timago.com](http://www.timago.com). Thank you for choosing Timago!

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