REF SIT-EXPERT

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BRUGSANVISNING

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USER MANUAL

Air adjustable cushion for wheelchair

Product code: SIT-EXPERT

1. Characteristics

Air adjustable cushion consists of an air chamber with self-inflating control valve. The cushion is equipped with removable cover with mounting strap made of two different kinds of fabrics. The upper part of the cover is composed of vapour permeable and moisture resistant material while the lower part consists of anti-slip fabric which prevents the cushion from moving on the seat surface. Owing to the control valve the cushion self-inflates. The cushion also adapts to user's body thanks to the adaptable air chamber.

2. Application

Air adjustable cushion is designed for patient vulnerable to development of bed sores. The cushion is recommended to be used in case of patients who are permanently in sitting position and who suffer from blood flow disorders in the peripheral circulation system. It is used in the preventive anti-decubitus treatment and in initial treatment of pressure sores.

3. Before first use

Step 1. Put the cushion on the seat with anti-slip side facing downward.

Step 2. The opening with the control valve should be placed at the front of the seat.

Step 3. Remove all the items from the cushion.

Step 4. Pull the valve out through the opening in the front of the cushion and ensure that its fully pulled out.

Step 5. Open the valve by turning it counter-clockwise and lifting it slightly upward. The cushion should self-inflate.

Step 6. Once inflated, close the valve by turning it clockwise.

CUSHION ADJUSTMENT

Step 1. Sit on the inflated cushion.

Step 2. Open the valve slowly until the air starts to escape. You can adjust the air flow additionally:

a) Press the cap to reduce air flow.

b) Lift the cap upwards to increase the air stream.

Step 3. Release the excess of air until the user sitting stably and comfortably.

Step 4. Once adjustment is complete, close the valve.

REMOVING AND PUTTING ON THE COVER

To remove the cover from the cushion, open the valve, unzip the cover and gently remove it from the cushion. To put the cover back on, open the valve and put the cover on from the side of the valve. Then, gently lead the valve through the opening in the cover and zip it. Inflate and adjust the cushion. (see: CUSHION ADJUSTMENT)

Note! Recommended thickness of the cushion should not be less than 1,5 - 2 cm.

4. Technical data

Lenght	43 / 46 cm
Depth	40 cm
Height	8 cm
Max. capacity	150 kg
Weight	0,80 / 0,90 kg

5. Notes

Timago International Group does not bear responsibility of improper moving in the product, failure to comply safety regulations and misuse.

6. Maintenance

- **Cleaning the cushion:** Clean the cushion with damp cloth using mild cleaning agents and prevent it from getting wet. Dry thoroughly after each cleaning. Usage of strong detergents may lead to damage of the product. Difficult stains may be removed with local hand washing.
- **Cleaning the cover:** The cover may be washed in washing machine at 40°C with the use of mild detergents. Before cleaning zip the cover. The cover may be dried with stream of hot air or in dry machine in low temperature.
- **Disinfection:** Disinfect the cushion using a cloth moistened in a mild detergent.

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!

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