# USER MANUAL

## Bathtub transfer bench

Product code: TGR-R TW 330


### 1. Characteristics

The transfer bench is made of high-strength, moisture-resistant PVC. It consists of a seat, an ergonomic handle and non-slip adjustable feet.

#### Elements of the bench:

- A. Seat
- B. Handle
- C. Non-slip adjustable feet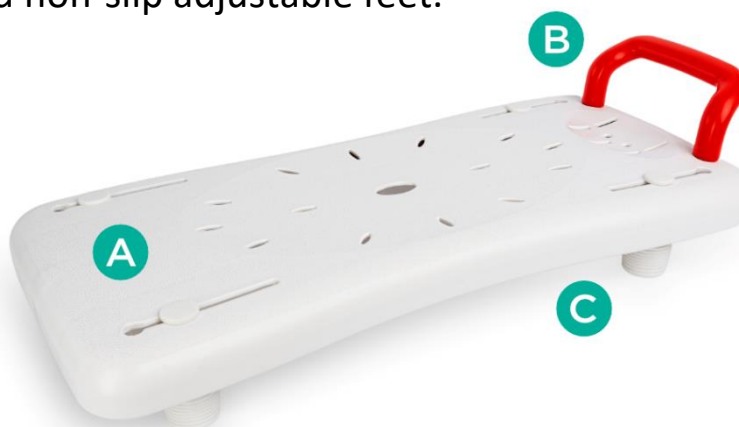

### 2. Application

The transfer bench is ancillary equipment that helps the user perform physiological activities. The chair is recommended for people with reduced mobility, which prevents or hinders the unassisted performance of daily living activities.

### 3. Assembly

**Step 1.** Assemble the handle and feet to the seat.

**Step 2.** Adjust feet so that they match with width of the bath.

**Step 3.** Tighten feet afterwards.

**Note:** Before first use, ensure the transfer bench is assembled correctly.

### 4. Technical data

Overall width	70 cm
Overall depth	41 cm
Seat dimensions	42 cm x 29 cm
Bench weight	2,8 kg
Maximum capacity	160 kg

## 5. Notes

**Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.**

## 6. Maintenance

- Clean the transfer bench regularly using only water and mild detergents for plastic materials. Dry thoroughly.
- **Do not use for cleaning:** strong detergents, abrasive agents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- Systematically make a visual inspection of all the components of the product including: tightening of feet, the condition of the seat and handle.

## 7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

## 8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

## 9. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction

The manufacturer checked for compliance with essential medical devices requirements.

Our complete line of products is available from our distributors. Learn more at: [www.timago.com](http://www.timago.com). Thank you for choosing Timago!

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