USER MANUAL

Aluminium wheelchair

Product code: T101

1. Characteristics

The wheelchair is made of a lightweight aluminum frame reinforced with a crossbar. It features a nylon-cotton seat with an additional cushion and a safety belt for the user. The wheelchair is equipped with a quick-release system, anti-tip wheels, a folding backrest, contoured flip-up armrests, attendant brakes, and swing-away footrests. It comes with either pneumatic or solid 24" rear wheels with handrims, as well as 8" solid rubber front wheels. For patients after amputations, it is possible to adjust the center of gravity on both the front and rear wheels. The wheelchair is also equipped with attendant brakes for the caregiver.

The set includes: frame with seat, footrests with footplates, protective straps, safety belt, assembly keys.

Elements of the wheelchair:

- A. Handgrip
- B. Attendant brake
- C. Armrest
- D. Armrest flip-up handle
- E. Parking brake
- F. Handrim
- G. Quick release function
- H. Footrest
- I. Footrest lever
- J. Adjustment of the center of gravity
- K. Anti-tip wheels
- L. Backrest folding mechanism

2. Application

The wheelchair is designed for people who have problems with unassisted moving and for patients with dysfunctions and reduced mobility. This self-propelled wheelchair can also be pushed by an accompanying person. The wheelchair is suitable for indoor and outdoor use.

3. Assembly

Step 1. Unfolding the wheelchair

Place the wheelchair on the ground. Unfold the wheelchair using the handgrips located at the backrest. Then, press the edge of the seat so that all the elements fit together.

Step 2. Assembly of footrests

Assemble the right and the left footrest. Put the assembly element into the opening in the front part of the frame and turn the footrest inwards until the footrest lever is locked. The footrest plates are equipped with straps to prevent feet from slipping off the footrests.

Step 3. Adjustment of footplates

Adjust the right and the left footplate by tightening the screw in the suitable mounting hole. Proper space should be left between the footplates and the ground as footplates set too low may cause difficulties in overcoming such obstructions as curb. Adjust the footplates so that the angle between trunk and thighs is 90° (optimum sitting position is provided to the user, then).

Step 4. Adjustment of anti-tip wheels

In order to adjust anti-tip wheels, located at the back of the wheelchair, pull the lock, move the wheels upwards or downwards, and select appropriate height.

Step 5. Checking the assembly of wheels

Lift the wheelchair slightly and spin the right wheel and then the left one by pulling the handrims and check if the wheels spin properly.

The wheelchair is ready to use.

FOLDING

The wheelchair can be re-folded in order to be transported and stored more easily. Swing away the footplates and set them in vertical position or remove. Detach the seat and backrest upholstery with bag. Lift the seat upwards by grabbing it from back and front until the frame of the wheelchair is folded.

FLIP-UP FUNCTION OF ARMRESTS

- To flip-up the armrest, press down the handle located in the front part of the armrest and swing it away.
- To close the armrest push it downwards and then pull the lock upwards.

PUMPING THE WHEELS

- If wheelchair is equipped with pneumatic or solid rear wheels 24" in diameter and swivel, PU front wheels 8" in diameter (tubeless tire).

Remember to control the air pressure in both wheels (version with pneumatic wheels) and check the condition of the tires in search of abrasions, cuts or surface depletion.

- To balance the air pressure in the wheels, twist off the cap of the valve, insert the pump into the valve (included in the set), and pump the air.

FOLDING BACKREST

- To fold the backrest, simultaneously pull downwards the folding mechanisms located under the handgrips on the back.

PARKING BRAKES

To lock the wheels, push the lever forward until the wheel is fully blocked (separately for left and right wheel). Please, note that the parking brake only serves to lock the wheels when the wheelchair is at a complete stop.

PARKING BRAKE ADJUSTMENT

The correct lock of the parking brake depends on its adjustment and position. The handle of the wheel lock should be positioned so that the brake shoe blocks the wheel firmly and the lever does not face excessive resistance.

- In case of excessive resistance, pull the handle slightly forward by twisting off the locking screws which are located on the parking brake. Set the brake shoe in such position that it blocks the wheel easily without excessive resistance.
- When the parking brake shoe does not touch or lock the wheel sufficiently, follow the above instruction by moving the handle in the opposite direction.

SEAT BELT

The belt, which is included in the standard equipment, protects the trunk in cases when the user slides down from the wheelchair or is not able to keep correct position independently.

ATTENDANT BRAKES

The levers of the brakes are located behind the backrest, under both grips.

- In order to stop the wheelchair, squeeze the lever of the brake handle. To release the brake, let go of the levers. Perform this activity smoothly and with appropriate force.
- The brakes are equipped with parking locks. In order to use them, squeeze the top lever of the brake handle till you hear “click” sound. To release parking brakes, squeeze the bottom lever of the brake handle.

ADJUSTMENT FOR PATIENTS AFTER AMPUTATIONS IN THE LOWER LIMBS AREA

Note! For professional adjustment, ask your retailer or other person who has appropriate knowledge and qualifications. Do not adjust inclination angle of the seat by yourself.

- To adjust the center of gravity, unscrew fixing screw in right and left wheel. Then, set the seat inclination by moving fixing axis of the wheel upwards or downwards, the same in both wheels.
- The seat inclination should be set in such a way so that the user does not slide down from the wheelchair and can freely use handrims.

4. Technical data

| Parameter | Version with pneumatic wheels | Version with solid wheels |
|-----------------------------|-------------------------------|-------------------------------|
| Overall width | 61/63/65/67/69 cm* | 61/63/65/67/69/76 cm* |
| Inner width | 41/43/46/48/51 cm* | 41/43/46/48/51/56 cm* |
| Overall depth | 107 cm | 107 cm |
| Seat height from the ground | 50 cm | 50 cm |
| Overall height | 95 cm | 95 cm |
| Seat dimensions | 41/43/46/48/51 x 42 cm* | 41/43/46/48/51/56 x 42 cm* |
| Weight | 15,5/16/16,5/17/17,5 kg* | 15,5/16/16,5/17/17,5/18,5 kg* |
| Type of crossbar | single crossbar | double crossbar |
| Max. capacity | 125 kg | 136 kg |
| Folded width | 30 cm | 30 cm |
| Frame colour | black | black |

* depends on seat width

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Maintenance

Carry out the maintenance of the wheelchair components regularly:

- The upholstery and the cushion can be cleaned with damp (not wet) cloth and cold or warm water, without the use of detergents. Difficult stains can be cleaned with a sponge. Do not soak the upholstery too much. Dry away from heat sources.
- Plastic elements should be cleaned with damp cloth possibly with the use of appropriate detergents for plastic elements. In such case follow the instructions given by the producer and safety measures.

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

| | | | |
|---|--|---|-----------------------------|
|  | Reference number |  | Manufacturer |
|  | LOT number |  | UDI |
|  | Serial number |  | Medical device |
|  | Note |  | Please read the instruction |
|  | The manufacturer checked for compliance with essential medical devices requirements. | | |

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!

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