USER MANUAL

Fixed commode chair

Product code: TGR-R KT-S 668

1. Characteristics

The commode chair is made of a powder-coated steel frame. It features legs with anti-slip rubber tips. The chair is equipped with a toilet seat and a removable plastic toilet bucket with a lid. It can be adjusted to the patient's needs thanks to its height adjustment, while the armrests provide stability and ensure safe use.

The set includes: side frames with armrests and legs, backrest tube, front and back frame, toilet seat with lid, toilet bucket with lid, mounting screws.

Elements of the commode chair:

- A. Side frame with armrest
- B. Backrest tube
- C. Back frame
- D. Front frame
- E. Toilet bucket
- F. Height adjustment
- G. Anti-sliding tips

2. Application

The commode chair is an ancillary device designed to assist users with personal hygiene activities. It is recommended for individuals with mobility impairments or reduced physical function, who experience difficulty walking independently or are unable to sit down and stand up without assistance. The chair serves as a toilet, thanks to the removable toilet bucket located beneath the seat. It facilitates the performance of physiological functions and helps caregivers provide effective assistance.

3. Assembly

UNFOLDING AND FOLDING

Step 1. Assemble the side frames with backrest tube using mounting screws.

Step 2. Fix the front and back frame to the side frames using mounting screws.

Step 3. On the bottom rail of the back frame mount the toilet seat with lid until you hear “click” sound of plastic grips.

Step 4. When using the commode chair as a support for a toilet seat, the toilet bucket or a sanitary cover should be attached to the holders of the frame.

HEIGHT ADJUSTMENT

- To adjust the height of the chair press the locking pin, select appropriate height of the leg and then lock it again, repeating steps on the other legs. The legs are adjustable in 2.5 cm increments.

Note! All legs must be set on the same height level.

USAGE

- Lift the toilet lid and remove the bucket lid. After use, cover the bucket with the lid and lower the toilet lid.
- In case of using the chair as a stand for the toilet seat, use the sanitary cover.

4. Technical data

Overall width	55 cm
Inner width	44 cm
Overall depth	47 cm
Seat height from the ground	43-58 cm
Overall height from the ground	64-79 cm
Seat dimensions	33 x 37 cm
Chair weight	4,8 kg
Maximum capacity	110 kg
Frame colour	grey

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Maintenance

Carry out regular maintenance of the technical condition of the chair:

- Clean the commode chair regularly using only water and mild detergents. Dry thoroughly.
- **Do not use** for cleaning: strong polishing agents and detergents, abrasive agents bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- Empty and disinfect the toilet bucket and the toilet seat after each use. Use toilet cleaning agents according to producer's instructions.

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: **www.timago.com**. Thank you for choosing Timago!

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