USER MANUAL

Folding commode chair with quick-release

Product code: TGR-R KT 618

1. Characteristics

The commode chair is made of a powder-coated steel frame. It features height-adjustable legs with anti-sliding rubber tips. The chair is equipped with a toilet seat with lid and a removable plastic toilet bucket with lid. Thanks to the quick-release system, the commode chair is easy to assemble. Its height can be adjusted to meet the patient's individual needs, while the armrests provide support and stability.

The set includes: chair frame, front and curved frame, toilet seat with lid, toilet bucket with lid

Elements of the commode chair:

- A. Backrest frame
- B. Front frame
- C. Toilet bucket
- D. Height adjustment
- E. Anti-sliding tips

2. Application

The commode chair is ancillary equipment which helps the user to perform hygienic activities. The chair is recommended to people with dysfunctions and reduced mobility who have problems with unassisted walking and for those who lost the ability of unassisted sitting down and standing up. The chair functions as a toilet owing to the toilet bucket located under the seat. The chair facilitates performing physiological activities and taking care of the user.

3. Assembly

Unfolding

Step 1: Unfold the chair frame. Join the left and right front frame (B) using the quick-release mechanism, then insert the backrest frame (A) into the mounting openings at the back till you hear click sound.

Step 2: Attach the toilet seat with lid to the bottom rail of the backrest frame until the plastic clips click into place.

Step 3: If the commode chair is used as a stand for a toilet seat, the toilet bucket or sanitary cover should be attached to the steel holders beneath the seat.

HEIGHT ADJUSTMENT

- To adjust the height of the chair press the locking pin, select appropriate height of the leg and then lock it again, repeating steps on the other legs. The legs are adjustable in 2,5 cm increments.

Note! All legs must be set on the same height level.

USAGE

- Lift the toilet lid and remove the bucket lid. After use, cover the bucket with the lid and lower the toilet lid.
- In case of using the chair as a stand for the toilet seat, use the sanitary cover.

Note! Remember to empty the toilet bucket after each use.

4. Technical data

Overall width	57 cm
Inner width	45 cm
Overall depth	50 cm
Seat height from the ground	40-54 cm
Overall height from the ground	66-80 cm
Seat dimensions	34 x 37 cm
Chair weight	5,3 kg
Maximum capacity	135 kg
Frame colour	white

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Maintenance

Carry out regular maintenance of the technical condition of the chair:

- Clean the commode chair regularly using only water and mild detergents. Dry thoroughly.
- **Do not use** for cleaning: strong polishing agents and detergents, abrasive agents bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- Empty and disinfect the toilet bucket and the toilet seat after each use. Use toilet cleaning agents according to producer's instructions.

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: **www.timago.com**. Thank you for choosing Timago!

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